Pump In Style® Advanced

EN Instructions for use | ES Instrucciones de uso | FR Mode d'emploi

PLEASE READ AND SAVE THESE INSTRUCTIONS. FOR MOST CURRENT BREAST PUMP INSTRUCTIONS VISIT WWW.MEDELABREASTFEEDINGUS.COM

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Important Safeguards

When using electrical products, especially when children are present, basic safety precautions should always be followed.

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

DANGER: To reduce the risk of electrocution:

- Always unplug electrical product immediately after use.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Do not reach for a product if it has fallen into water. Immediately unplug from electrical outlet.

WARNING: To avoid fire, electrocution, or serious burns:

- Do not leave product unattended when plugged into an electrical outlet.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Use the product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Keep cord away from heated surface.
- Never use while sleeping or when drowsy.
- Never drop or insert any object into any opening or tubing.
- Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
- Always inspect power adaptor and battery pack wires prior to use for damage or exposed wire. If damage is found, immediately discontinue use of power adaptor or battery pack and call Medela Customer Service at 1-800-435-8316.

The warning symbol identifies all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breast pump. When used in conjunction with the following words, the warning symbols stand for:

WARNING Can lead to serious injury or death.

CAUTION Can lead to minor injury.

NOTE Can lead to material damage.

INFO Useful or important information that is not related to safety.

**WARNING: To avoid health risk and reduce the risk of injury:**

- This product is intended for use by a single user only and should not be shared between users.
- Do not drive while pumping.
- Never use while pregnant, as pumping can induce labor.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use.
- Wash all parts that come in contact with your breast and breast milk after every use.
- Inspect all appropriate pump components before each use.
- Do NOT continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- If tubing becomes moldy, discontinue use and replace tubing.
- If breast milk backup occurs, you must clean tubing, faceplate and diaphragm before your next pumping session.

**CAUTION: Can lead to minor injury:**

- Do NOT wrap cord around adaptor body.
- Use only the power adaptor that comes with the Pump In Style® Advanced Breast Pump.
- Use only the Pump In Style Advanced battery pack that comes with the Pump In Style Breast Pump.
- Pump only with the breast pump in an upright position.
- Do NOT shorten tubing.
- Make sure the voltage of the power adaptor is compatible with the power source.
- Plug the power adaptor into the breast pump first and then into the wall socket.
- Do NOT use antibacterial or abrasive cleaners/detergents when cleaning breast pump or breast pump parts.
- Never put breast pump motor in water or a sterilizer, as you can cause permanent damage to the breast pump.
- Do not attempt to remove the breast shield from your breast while pumping. Turn the breast pump off and break the seal between your breast and breast shield with your finger, then remove breast shield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.

IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

SAVE THESE INSTRUCTIONS

Table of Contents

Important Safeguards	2-3
Product Description	6-7
1. Cleaning Instructions	8-15
1.1 Before using for the first time	8
1.2 After each use.....	8-9
1.3 Sanitize daily	10-11
1.4 Tubing, faceplate and diaphragm cleaning	12-14
1.5 Cleaning your breast pump bag	15
2. Assembly	16-23
2.1 Assembly of pump kit	16-19
2.2 PersonalFit™ Sizing	20
2.3 Powering your pump.....	21-23
2.3.1 Power adaptor care	21
2.3.2 Battery pack	22
2.3.3 Traveling outside of the U.S.	23
3. Pumping	24-27
3.1 Device operation.....	24-27
4. Storing Breast Milk.....	28
4.1 Storing breast milk in your Pump In Style® Advanced cooler bag.....	28
5. Preparing & Feeding Breast Milk	29
5.1 Preparing breast milk	29
5.2 Feeding breast milk.....	29
6. Troubleshooting.....	30
7. Supplemental Information	31
8. Disposal.....	32
9. Warranty	33

Take a moment to read through this entire instruction manual before using this product for the first time.

PLEASE SAVE THESE INSTRUCTIONS.

It is best to wait until your breastfeeding routine is established (approximately 4 weeks) before expressing breast milk, unless otherwise advised by your healthcare professional.

Glossary of Terms

2-Phase Expression® technology – research based technology that mimics a baby's natural nursing rhythm.

Expression Phase – slower sucking/pumping rhythm for gentle and efficient milk removal as quickly as possible.

Maximum Comfort Vacuum™ – highest vacuum setting where a mother still feels comfortable during pumping. It is different for every mother.

Stimulation Phase – fast sucking/pumping rhythm to stimulate the milk ejection reflex and to start the milk flowing.

Intended Use

For lactating women to express and collect milk from their breasts to complement breastfeeding.

If you have medical reasons or other needs for exclusively pumping, it is recommended that you use a hospital grade breast pump such as our Symphony® Breast Pump with 2-Phase Expression technology. To learn more, visit us at www.medelabreastfeedingus.com.

Product Description

Pump In Style® Advanced (PNSA) is a personal use electric breast pump that includes 2-Phase Expression technology and is capable of single and double pumping.

Product Description

Medela reserves the right to substitute any component or accessory with a replacement of equivalent performance.

2x PersonalFit™ Breast shields 24 mm

(for other breast shield sizes see **Section 2.2**, page 20)

Item 87073

2x Connectors


Item 87071 or 8107264

2x Valves

Item 87089

2x Membranes

Item 87088

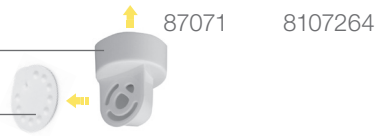4x Bottles

Item 6100050

Replacement 87130

4x Lids

Item 8107183

Replacement 87165

1x Battery pack

Item 9017002

Replacement 67553

2x Tubing

Item 87212

1x Power adaptor

Item 9207010

Replacement 68030

1x Cooler bag

Replacement 67068

(includes ice pack & 4 bottles with lids)

1x Ice pack

Item 8117010

Replacement 87092


On-the-go Tote

Backpack

1x Faceplate
Item 6007132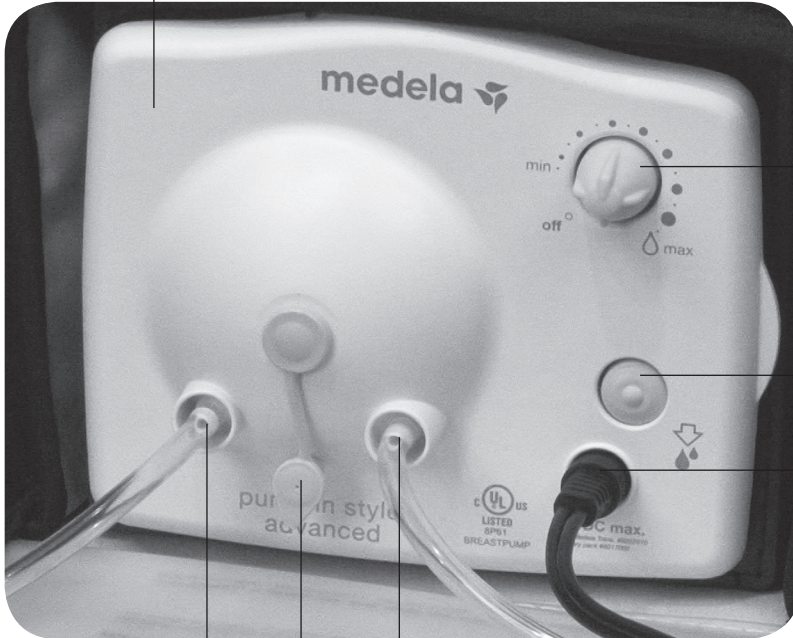

Tubing port

Port plug

Tubing port

On/Off button
Increase/reduce vacuum

Let-down button

Power adaptor port

Additional parts included with The Metro Bag™

Pump In Style®
Advanced
The Metro Bag™

27 mm PersonalFit™ Breast shields..... 87274

Metro Bag including:


Flexible, Plastic Workspace 3007299

2 Mesh Storage Bags

1. Cleaning

1.1 Before using for the first time

Clean and sanitize the parts below before using your pump for the first time. Follow the cleaning and sanitizing instructions in **Sections 1.2 and 1.3** that follow.

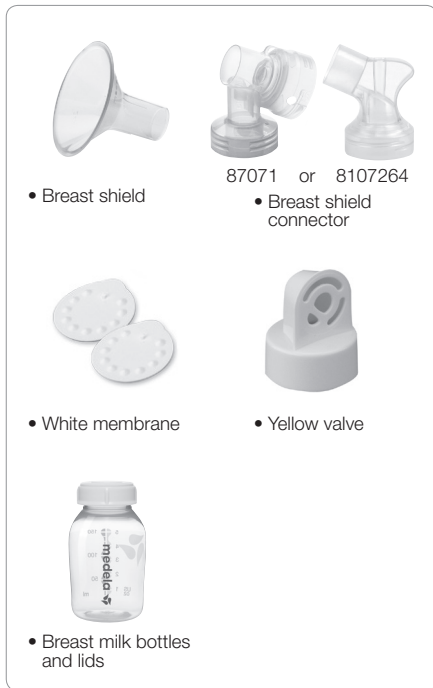1.2 After each use

Clean all parts that come in contact with your breast and breast milk. See image in **Section 1.1**.

Separate breast shields from breast shield connectors.

Rinse in cool water all separated parts that came in contact with breast and breast milk in order to remove breast milk residue.

Caution

- Only use drinking-quality tap or bottled water for cleaning.
- Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.

Supplies Needed:

- Mild Dish Soap
- Clean Sink or Bowl
- Clean Dish Towel or Soft Brush
- Quality Drinking Water

2

Twist and pull the yellow valves off the breast shield connectors.

3

Remove the white membranes from the yellow valves.

Washing Options:

Washing in sink:

5

- Soak all separated parts in warm soapy water for 5 minutes.
- Clean with a clean dish-cloth or soft brush.
- Rinse all separated parts with clear water.
- Allow all parts to air dry in a clean area.
- Store dry parts when not in use.

or

Washing in dishwasher:

- You may wash your kit parts by following the instructions on the Quick Clean™ Breastmilk Removal Soap bottle. (Sold separately)
- Wash all separated parts on top rack of dishwasher.
- Allow all breast pump parts to air dry in a clean area.
- Store dry parts when not in use.

1. Cleaning (cont.)

Note

- **Wash** breast pump kit parts after each use.
- **Sanitize** breast pump kit parts once daily.
- Take care not to damage parts of the pump kit during cleaning.
- If the individual components of the pump kit are cleaned in a dishwasher parts may be discolored by food pigments. This has no impact on the function.
- Store the dry pump kit in a clean bag/container until next use.

1.3 Sanitize daily

Wash hands thoroughly.

3

Allow water to cool and gently remove parts from water with tongs.

Separate all parts that come in contact with your breast and breast milk.

- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water.
- Let parts boil for 10 minutes.

or

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts when not in use. Do NOT store wet or damp parts.

You may sanitize your kit parts by following the instructions on Medela's Quick Clean™ Micro-Steam™ bag. (Sold separately)

(Do not clean tubing in a Micro-Steam bag.)

1. Cleaning (cont.)

Caution

- Do not store wet or damp parts as mold may develop.
- If tubing becomes moldy, discontinue use and replace tubing. To find replacement parts visit www.shopmedela.com

Note


- Tubing should only be washed if dirty or milk is present. Follow instructions in **Section 1.4**.
- Cleaning tubing is not necessary if condensation inside the tube is from previous washings or atmospheric conditions. Follow drying instructions in **Section 1.4**.

1.4 Tubing, Faceplate and Diaphragm Cleaning

Clean the faceplate monthly. Also, inspect tubing after each pumping session. Clean tubing, faceplate and diaphragm if dirty or there is any sign of breast milk.


Turn off breast pump.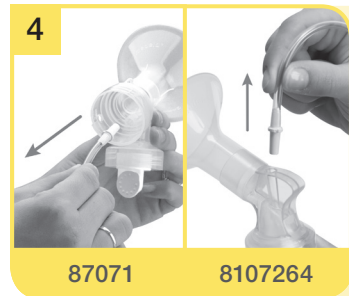

Remove tubing from breast shield.

2

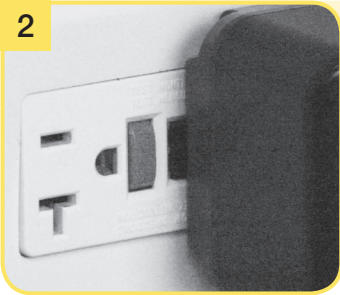Unplug breast pump from power source.

3

Remove tubing by pulling straight out of tubing ports. Do not wiggle or pull tubing at an angle.

5

Rinse tubing in cool water to remove breast milk.

6

- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry or connect the tubes to the running breast pump and allow any moisture that has formed to dry.

Continued next page.

1. Cleaning (cont.)

Note

- Make sure faceplate and diaphragm are completely dry before reseating.
- The faceplate must be seated correctly and snapped together at all connection points. This ensures the pump will have adequate vacuum and optimum performance.

1.4 Tubing, Faceplate and Diaphragm Cleaning (cont.)


Snap off the faceplate using the thumb tab.

Locate the 3 connection points on the reverse side of the faceplate and on the pump.

8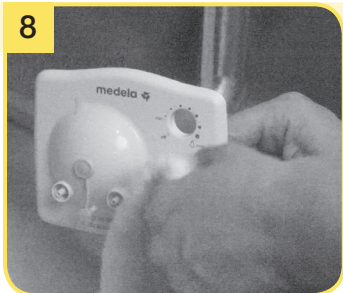

- Soak in warm soapy water for 5 minutes.
- Clean with a clean dish-cloth or soft brush.
- Rinse parts with clear water.
- Allow to air dry in a clean area.

9

- Some breast pump models may have a diaphragm that appears different than pictured above.
- Carefully wipe the entire surface area of the diaphragm where particles or liquid may have accumulated, with a clean (no soap), damp cloth.
- Allow all parts including edges to air dry, **completely**, in a clean area before replacing the faceplate.

1.5 Cleaning your breast pump bag

11

Align connection points and hold back of pump with one hand while pushing on the faceplate until it snaps into place.

Use a damp (not wet) cloth to wipe the breast pump bag after each use.

2. Assembly


Parts you will need for this step:

	
• Breast shield	87071 or 8107264 • Breast shield connector
	
• White membrane	• Yellow valve
	
• Breast milk bottles and lids	• Tubing
	
• Battery pack	or • 9 volt power adaptor
	
• Breast pump	or • Metro pump

2.1 Assembly of pump kit

Wash hands before touching breast pump, kit and breasts and avoid touching the inside of containers or lids.

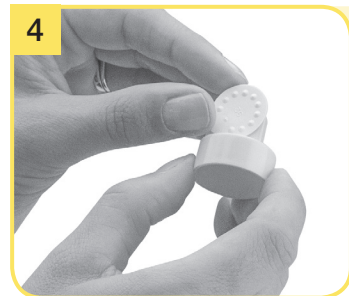Snap membranes onto valves until membranes lie completely flat on valve.

Note

- Inspect valves for cracks or chips and inspect membranes for holes, tears or warping. If you notice any damage to these parts, replace them immediately.

Caution

- Wash hands thoroughly with soap and water before touching the breast and pump parts.

2 Place your breast pump on a clean and dry surface. Ensure that the surface is solid and secure so your breast pump will not tip over or fall.

3 Securely attach breast shield to breast shield connector.

5 Push assembled yellow valves and white membranes onto bottom of breast shield connectors.

6 Screw bottles onto breast shield connectors.

Continued next page.

i Info

- Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

2. Assembly (cont.)


Warning

- Use only the power adaptor supplied with the Pump In Style® Advanced.
- Always inspect the 9 volt power adaptor and Battery Pack cords prior to use for damaged, kinked, twisted, frayed or exposed wires. If damaged, immediately discontinue use. For replacement parts visit www.shopmedela.com.

Twists, kinks or bends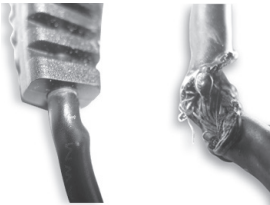

Has bulges


Exposed wires

Note

- See **Section 2.3.2** Battery Pack Assembly for instructions on the use of the Battery Pack.
- See **Section 2.3.3** Traveling Outside the U.S. for information on using your device internationally.

2.1 Assembly of Pump Kit (cont.)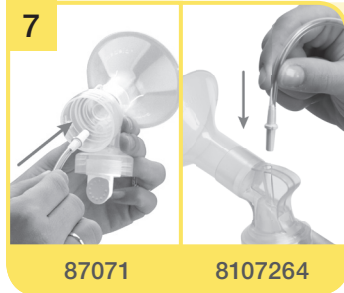

Insert hard yellow adapter ends of tubing into hole on the back of breast shield connectors.

Plug the metal fitting on end of 9 volt power adaptor or Battery Pack cord into the power jack on front of breast pump faceplate.

8

Double pumping: Insert free ends of tubing onto both ports on front of breast pump faceplate.

9

Single pumping: Insert free end of tubing onto one port on front of breast pump faceplate. Place port cap over the unused port.

11

Plug 9 volt power adaptor into a standard electrical outlet.

2. Assembly (cont.)

2.2 PersonalFit™ Sizing

1. Medela's PersonalFit sizing

Pumping should not hurt. For maximum comfort and pumping efficiency, Medela offers five breast shield sizes.

21
mm

24
mm

27
mm


30
mm

36
mm

Visit MedelaBreastshields.com to view the full guide to help you determine your optimal size based on your nipple diameter.

2. Test your breast shield size

1. Start with the 24 mm that came with your pump, or the size determined by measuring.
2. Center nipple and gently hold breast shield against your breast.
3. Adjust for Maximum Comfort Vacuum™, to achieve optimum suction level.
4. Refer to images while pumping in expression.

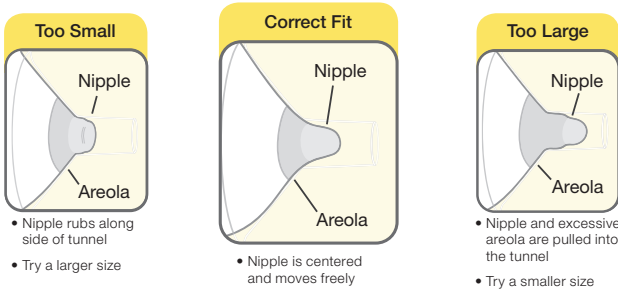3. Should you try a new size?

- Does your nipple rub sides of tunnel, to the point of causing discomfort?
- Do you see excessive areola being pulled into tunnel?
- Do you see any redness?
- Is your nipple or areola turning white?
- Do you feel unexpressed milk after pumping?

If you answered **“YES”** to any of these questions, consider trying a new size by following the measuring instructions above.

If you are still unsure if you selected the correct size, see a lactation consultant, breastfeeding specialist, or visit MedelaBreastshields.com for assistance in choosing the right size breast shield.

Caution

- While some discomfort may be felt when first using a breast pump, using the breast pump should not cause pain.
- If you are experiencing discomfort at the base of the nipple due to rubbing of your breast tissue with the breast shield tunnel, use of a lubricant such as Tender Care™ Lanolin may be beneficial.
- For assistance with correct breast shield sizing and comfort please visit www.medelabreastshields.com or see a lactation consultant / breastfeeding specialist.

2.3 Powering your pump

2.3.1 Power adaptor care


To care for your adaptor be sure to follow these storage directions.

Do not wrap cord of power adaptor around the plug body.

Correct!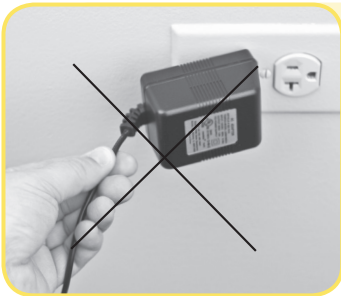

Do not unplug power adaptor by pulling on the cord.

Correct!

2. Assembly (cont.)

! Note

- Remove the batteries if the Battery Pack is not going to be used for an extended period of time.
- If breast pump does not turn on, make sure the batteries have been inserted properly. If this does not solve the problem refer to the troubleshooting section of this instruction booklet or contact Medela Customer Service at 1-800-435-8316.
- Each set of batteries will provide approximately 2 hours of pumping time.
- Do not mix old and new batteries.
- Do not mix alkaline, standard (carbon-zinc) or rechargeable batteries.
- For information about traveling with your breast pump and expressed breast milk, visit www.TSA.gov.

2.3 Powering your pump (cont.)

2.3.2 Battery pack

Use 8 AA rechargeable batteries or alkaline batteries.

Replace battery cover.

2

Replace all batteries at the same time.

Slide off both battery covers. Four (4) batteries are placed on each side of battery pack.

3

Place the NEGATIVE (-) end of the battery against the front of the spring by pushing the battery in and down in one motion until the battery snaps into place.

The POSITIVE (+) end of the battery should be near the positive sign on the battery pack.

2.3.3 Traveling outside of the U.S.

When traveling outside of the U.S., there are 2 power options:

Use the Battery Pack that was included with your Pump In Style® Advanced breast pump.

or

Purchase the 9 volt Portable Vehicle Adaptor (sold separately). Visit our online store at www.shopmedela.com.

The Portable Vehicle Adaptor may not work with all vehicles. Please check the owner's manual for compatibility with 12-24 volt input.

3. Pumping

Caution

- Always inspect breast shields, breast shield connectors, white membranes, yellow valves, and tubing prior to use for cleanliness. If dirty, see **Section 1.2**.
- If you experience pain or other symptoms while pumping seek advice from your healthcare professional or breastfeeding specialist. See **Section 7** for more information.
- Do not try and express with vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma may decrease milk output.
- Make sure tubing is not kinked or pinched while pumping.
- Contact your healthcare professional or breastfeeding specialist if you can express only minimal or no milk, or if expression is painful.

Info

- Research has shown that you will pump more efficiently – get more milk in less time – when pumping at Maximum Comfort Vacuum™ during the Expression Phase.
- You should reassess your Maximum Comfort Vacuum throughout your pumping experience, because it can change during the different stages of lactation, **see step 5 in Section 3.1**.
- Do not tilt or overfill containers when pumping.

3.1 Device Operation


Wash hands before touching breast pump, kit and breasts and avoid touching the inside of containers or lids.

After two minutes, the pump will automatically change to the Expression Phase.

- a. If you let-down (your milk starts to flow) sooner than 2 minutes, press the let-down  button.
- b. If let-down has not happened during the Stimulation Phase and the pump has switched to the Expression Phase, press the let-down  button once to return to the Stimulation Phase.

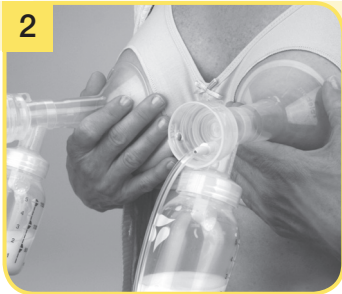
Center the assembled breast shields over your nipples.

To begin pumping, turn dial clockwise from the off position. Your pumping session will start in the Stimulation Phase. While pumping, adjust the speed/vacuum by turning the dial to your comfort level.

To find your Maximum Comfort Vacuum, increase speed/vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.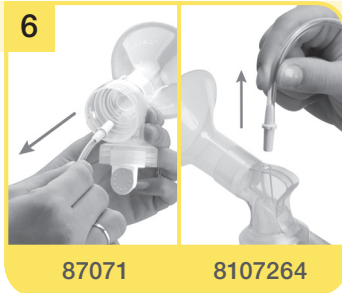

When your pumping session is over, unplug tubing from the back of the breast shield(s) before setting down the bottles.

Continued next page.

3. Pumping (cont.)

! Note

- Always disconnect Pump In Style® from the power source after expression.
- Only fill the bottle to the 150 mL mark.

i Info


- **How often should you pump?** A breast pump is a replacement for when you are separated from your baby. It is important to pump when the baby would be breastfeeding, which usually means a working mother pumps 2-3 times during an 8 hour working day.
- **How long should your pumping session last?** Pumping times can vary from mother to mother sometimes 15 minutes, sometimes up to 30 minutes.
- **How should your breasts feel after pumping?** After pumping, your breasts should feel soft and there should be no firm areas. Before pumping, your breasts will have a firm, heavy feeling. After pumping make sure your breasts are not firm or have lumpy areas. This could indicate that the breast is not draining all over. See **Section 7** for more information.

3.1 Device Operation (cont.)

Close the bottle with a lid.

- Follow the instructions in Section 4 “Storing Breast milk & Section 5 “Preparing and Feeding Breast milk”.

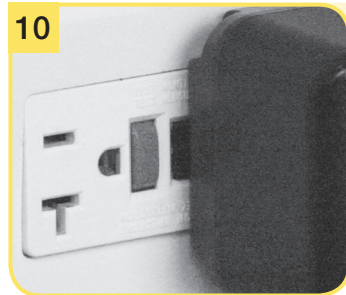
Unplug breast pump from power source.

8 Allow the pump to continue to run while you store your breast milk. This will allow any condensation that has formed from the natural results of humidity to dry in the tubing.

9 Turn off breast pump.

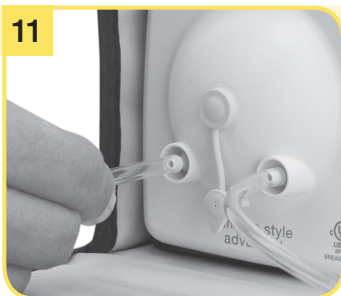11 Prior to storing your pump, remove tubing by pulling straight out of tubing ports. Do not wiggle or pull tubing at an angle.

12 Disassemble and clean per Cleaning Instructions in **Section 1**.

4. Storing Breast Milk

Warning

- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- Do not microwave breast milk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breast milk.)

Note

- If the ice pack is frozen solid, it will safely cool your breast milk for up to 10 hours in a room temperature environment.

4.1 Storing breast milk in your Pump In Style® Advanced cooler bag

The Pump In Style Advanced cooler bag requires one contoured ice pack (#87092, included) for cooling up to four 5 oz bottles of breast milk.

- As soon as you are finished pumping, place the breast milk bottles or breast milk storage bags in the cooler bag with the ice pack.
- Transfer breast milk bottles or breast milk storage bags to the refrigerator or freezer once you are home.

5. Preparing and Feeding Breast Milk

Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)			
Room Temperature	Cooler with Ice Pack	Refrigerator	Freezer
4–6 hours at 66–78 °F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–-20 °C)

References: www.BreastmilkGuidelines.com

5.1 Preparing breast milk

- Thaw breast milk overnight in the refrigerator. Thawed breast milk is safe in the refrigerator for 24 hours. Do not refreeze thawed breast milk.
- Quickly thaw breast milk by holding the bottle under warm running water.
- Place the sealed bottle in a bowl of warm water for 20 minutes to bring it to body temperature.
- If you are adding expressed breast milk to a container of already frozen breast milk, make sure to add a lesser amount than the already frozen amount.

5.2 Feeding breast milk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, nipple and other components immediately before and after each use. If nipple appears cracked or torn, discontinue use immediately.
- To prevent possible choking hazard, test strength of nipple by pulling on bulb portion of the nipple.
- Do not attempt to enlarge the nipple hole.
- Infants must not be bottle fed without adult supervision.
- Nipple should not be used as a pacifier.

For additional breast milk collection and storage information, please visit:

www.BreastmilkGuidelines.com

6. Troubleshooting

<p>Low or No Suction</p>	<p>Valve and Membrane Check</p> <ul style="list-style-type: none"> • Remove and separate valve from membrane • Inspect the valve for chips or cracks. Inspect the membrane for holes or tears. If flawed or damaged replace valve and/or membrane prior to pumping • Wash valve and membrane per Cleaning section • Reassemble. Make sure the membrane lays flat on the valve, and if it does not, replace the valve and/or membrane prior to pumping <p>Breast shield and Breast shield Connector Check</p> <ul style="list-style-type: none"> • Inspect breast shield and breast shield connector to make sure there are no cracks and they securely attach to each other <p>Faceplate Check</p> <ul style="list-style-type: none"> • Wash faceplate and backplate per Cleaning section 1.4 steps 1-11. • Reassemble faceplate and backplate, and reattach tubing to faceplate port <p>Connection Point Check</p> <p>Inspect following connection points to ensure attachment is secure</p> <ul style="list-style-type: none"> • Tubing adaptor to breast shield connector • Tubing end to faceplate port • Faceplate to backplate • Test vacuum. If suction is not improved, contact Medela Customer Service at 1-800-435-8316.
<p>Audible air coming from port cap/faceplate</p>	<p> CAUTION: Do not attempt to plug hole in cap or port with additional material.</p> <ul style="list-style-type: none"> • There is a built-in air leak for vacuum control, some audible air is normal.
<p>Power source</p>	<ul style="list-style-type: none"> • If pump is not turning on with the power adaptor try the battery pack. • If the pump is not turning on with the battery pack, be sure the batteries have been inserted correctly. • If pump is still not working with the power adaptor or the battery pack, contact Medela Customer Service at 1-800-435-8316.
<p>Faceplate port broken/ cracked</p>	<p> CAUTION: If you are feeling a loss of suction with the broken faceplate, do not use the pump until the new faceplate arrives.</p> <ul style="list-style-type: none"> • Contact Medela Customer Service at 1-800-435-8316 or order online at www.shopmedela.com.
<p>Condensation or milk in tubing</p>	<ul style="list-style-type: none"> • Refer to “Tubing and faceplate cleaning” section 1.4.
<p>No “let-down” or breast milk expression</p>	<ul style="list-style-type: none"> • Ensure your breast pump is assembled correctly and that there is suction. • Relax and take a 10-15 minute break if let down is not achieved after 2 consecutive pumping sessions. • Consult with your healthcare provider or breastfeeding specialist if expression does not occur.

7. Supplemental Information

Supplemental Information

Following are some common breastfeeding related conditions. **If you experience any of these symptoms, contact a healthcare professional or breastfeeding specialist.**

	Symptom	Potential Causes
Engorgement	Breasts are hard and uncomfortable, possibly with reddened areas.	Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.
Blocked/Plugged Ducts	Area of breast will look reddened and may be tender to touch.	Milk not being drained from a specific duct. The area becomes 'clogged' up and milk is then prevented from flowing.
Mastitis	It often follows on from engorgement but can just occur suddenly. The initial symptoms are similar to flu – fatigue, headache and muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.	A bacterial infection in the breast tissue which is often accompanied by cracked nipples.

A Healthcare Professional or Breastfeeding Specialist can give you guidance in breastfeeding your baby and using a breast pump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of Lactation Professionals are available at www.medelalocator.com or call 1-800-TELL-YOU (1-800-835-5968).

If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service at 1-800-435-8316 or email customer.service@medela.com.

8. Disposal

Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health.

Note

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Recycle your breast pump

Through the Medela Recycles program, Medela, Inc. provides customers the option to properly recycle their breast pump after they've finished their breastfeeding journey. For information on how to recycle your pump and to receive your prepaid shipping label, visit www.medelarecycles.com.

This program is currently only available to U.S. residents. We only accept the return of electric breast pumps. All other breast pump kit components (connectors, breast shields, tubing, bottles, valves, membranes, and cooling elements) can be disposed of in your waste management recycling container at your home. Cooler bags and other soft-good items are not recyclable.

9. Warranty

Caution

This is a single user product. Use by more than one person may present a health risk and performance questions and voids the warranty.

This product is warranted by Medela, Inc. to the original retail purchaser to be free from defects in material and workmanship for the period of one year for pump mechanism (90 days for parts and accessories) from the date of purchase. Warranty can only be claimed in the country of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement, parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

Before you make a claim under this warranty, it may save time and expense to call Medela Customer Service (toll free) at 1-800-435-8316. You may also call this number for additional information concerning this warranty. If you wish to make a claim under this warranty, you must return this product to Medela with a return authorization number received from Medela Customer Service, prepaid, together with your dated bill of sale or other proof of purchase and a brief statement of the problem to the following address:

Medela, Inc. – Returns, Door 4501
1101 Corporate Dr.
McHenry, IL 60050
ATTENTION: RETURNS

Call first for authorization number. Returns not accepted without an authorization number.

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Printed in the USA. Medela, 2-Phase Expression and Pump In Style are registered trademarks of Medela Holding, AG.
Quick Clean, Micro-Steam, Maximum Comfort Vacuum, The Metro Bag and PersonalFit are trademarks of Medela, Inc.
Medela Canada, Inc.,
Quick Clean, Micro-Steam, Maximum Comfort Vacuum, The Metro Bag y PersonalFit son marcas comerciales de Medela, Inc.